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


POST SKINPEN® PRECISION PROCEDURE GUIDELINES

It is important to keep your skin well hydrated after the SkinPen procedure. Microneedling causes a temporary increase in water loss so keeping your skin hydrated and protected during this period will aide your recovery experience.

1. Do not use any other products, including cleanser, on your skin within the first 24 hours post-procedure. Unexpected complications may occur when products not proven safe for use with microneedling are applied post-procedure. In lieu of cleansing your skin, you may rinse with cool water as needed.
2. If you have RESCUE Calming Complex, begin applying as needed after the first 24 hours to help soothe and calm the skin. RESCUE was specifically formulated for post-microneedling.
3. Strictly avoid sun exposure the day of the procedure as you will not be wearing sunscreen. You may return to sunscreen use 24 hours following the procedure. Continue to minimize sun exposure for 72 hours post-procedure.
4. Avoid strenuous exercise or excessive perspiration for 72 hours post-procedure, as excess blood flow and sweat can cause discomfort and irritation to compromised skin.
5. Avoid using any active skincare topical products, such as retinoids, vitamin C etc., for the first week after the SkinPen Precision procedure to reduce risk of irritation.
6. Schedule your next SkinPen Precision appointment with your healthcare provider as recommended.
7. Contact your health care provider with any questions regarding the procedure.

SKINPEN® PRECISION PROVIDER STAMP



YOUR NEXT
SKINPEN® PRECISION
APPOINTMENT :

____ / ____ / 20 ____

SkinPen
BY CROWN AESTHETICS



MedEnvoy
Prinses Margrietplantsoen 33
Suite 123
2595 AM The Hague
The Netherlands

Discontinue use of post-treatment products and call your doctor immediately if you have any concerns relating to your treatment response, or if you experience: 1. Signs of infection such as an increase in redness, warmth, itching, or pus formation, or 2. Allergic reactions to the test material (s). Rare allergic reactions can consist of severe contact dermatitis (rash) or itching, swelling, or difficulty breathing. This risk is increased for individuals with a history of allergies, and individuals with asthma and/or a history of hives, and itching. Indications for SkinPen® Precision may vary based on your country. Please visit www.skinpen.com/international